# *Pen Needles*

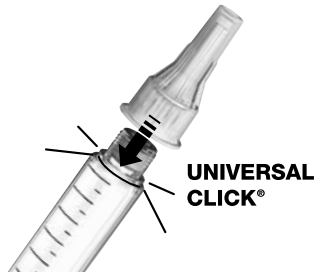
with Clickfine<sup>®</sup> technology

## **Instructions for use**

Tips for safe and comfortable injections inside!

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These pen needles are intended for the hypodermic injection of fluids into the body when attached to an injection pen.

**Universal Click®: Simply clicks on all these diabetes pens**

|                                   |                                      |
|-----------------------------------|--------------------------------------|
| <b>Apidra® SoloSTAR®</b>          | <b>Levemir® FlexPen®</b>             |
| <b>LANTUS® SoloSTAR®</b>          | <b>NovoLog® FlexPen®</b>             |
| <b>Humalog® KwikPen™</b>          | <b>NovoLog® Mix 70/30 FlexPen®</b>   |
| <b>Humalog® Mix50/50 KwikPen™</b> | <b>NovoPen® 3, 4 &amp; Junior</b>    |
| <b>Humalog® Mix75/25 KwikPen™</b> | <b>Victoza® Pen</b>                  |
| <b>Humulin N® Pen</b>             | <b>Byetta® Pen 5mcg &amp; 10 mcg</b> |
| <b>Humulin 70/30® Pen</b>         | <b>SymLinPen® 60 &amp; 120</b>       |
| <b>HumaPen® Luxura™ HD</b>        | <b>Autopen®</b>                      |
| <b>HumaPen® Memoir™</b>           |                                      |

**All brands listed are trademarks of their respective owners.**

**Follow pen's instructions for use.**

Consult your physician and/or health care professional and follow your pen's instructions for use before using these pen needles. Please contact us if you have any questions.

Remove pen needle seal.

Click pen needle onto pen, by pressing straight down using the outer protective cap\*.

Do not click on at an angle.

\* Note: The pen needle can also be twisted on; a secure fit is ensured even if there is no noticeable stop

Remove the outer protective cap by pulling it straight off; save it.

Remove the inner protective cap.

Penetrate skin quickly and inject slowly according to the instructions of your physician and/or health care professional and of the injection pen manufacturer.

Carefully replace the outer protective cap (if self-injecting only).

Unscrew the pen needle and safely dispose of it according to your physician's and/or health care professional's advice.

## **⚠ Warnings:**

---

### **⚠ AVOID INFECTIONS:**

- Do not use the pen needle if the seal is damaged, contaminated or has been opened by someone other than you.
- Do not use the pen needle after the expiration date on the packaging has expired.
- Before each injection, clean the injection site thoroughly according to your physician's and/or health care professional's instruction.
- Use each pen needle only once.
- Do not re-use, clean and/or re-sterilize pen needles.
- Carefully replace the outer protective cap on the used pen needle and dispose of it safely immediately after self-injection to avoid pricking yourself or other persons accidentally. Follow your physician's and/or health care professional's advice for proper disposal in your area.
- If these instructions are not followed, there is a risk of infection.
- Seek medical attention if you have any health concern.

## **Warnings:**

---

### **NEVER HANDLE SOMEONE ELSE'S PEN NEEDLE**

- **To avoid accidental needlesticks, which can transmit serious infections, never recap, remove, or destroy a pen needle that has been used by someone else.**

### **Specific directions for health care professionals:**

Professional health care personnel should follow the specific directions for health care personnel including, without limitation, OSHA, CDC, or local policy when using these pen needles, including directions for recapping and disposing. To obtain the specific directions, please consult a physician.

## **Warnings:**

---

### **GET THE CORRECT DOSAGE**

- Consult your physician for the right pen needle length for you.
- Prime your pen according to your pen's instructions for use before each injection to ensure that the needle allows free flow of medication. This will prevent incorrect dosage caused by air in the cartridge or a clogged needle. Use a new pen needle if priming is not successful.
- Inject slowly according to your physician's and/or health care professional's advice and pen's instructions for use and wait for a few seconds after medication has been completely discharged before withdrawing the pen from the injection site.
- Check after each injection according to your physician's and/or health care professional's instructions whether medication drips from the pen needle, pen, or injection site. If the medication drips, follow your physician's and/or health care professional's instructions regarding the handling of potential under-dosage.
- Seek medical attention if you have any health concern.

### **CHECK STEEL CANULA ATTACHEMENT**

---

- **Check after each injection to make sure the steel canula is still properly attached to the pen needle. If it is not properly attached, it can become stuck in the injection pen, inside the pen needle, in the body, or remain under the skin after injection.**
- **If the steel canula has been detached from the pen needle, follow your physician's and/or health care professional's instructions regarding the handling of potential under-dosage.**
- **Seek immediate medical attention if a detached steel canula becomes stuck in the body or remains under the skin or if you have any health concern.**

### **AVOID BENDING OR BREAKING THE NEEDLE**

---

- **Do not change the direction of the pen and pen needle after inserting the pen needle into the skin. Changing the angle can result in bending or breaking of the pen needle. A broken pen needle can remain stuck in the body or remain completely under the skin.**
- **Seek immediate medical attention if a broken pen needle becomes stuck in the body or remains under the skin.**

## TIPS FOR A COMFORTABLE INJECTION

- Change the injection site regularly in accordance with your physician's and/or health care professional's advice. If this is not done, tissue hardening may result.
- For most comfortable injection, inject medication at room temperature.
- Make sure no air bubbles remain in the pen before injection.
- Wait until topical alcohol (if used) has evaporated completely before injection.
- Keep muscles in the injection area relaxed when injecting.
- Penetrate the skin quickly.
- Do not change the direction of the pen needle during insertion into or withdrawal from the skin.
- Do not re-use pen needles.

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Can-Am Care LLC,